Nederlands

Demping

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APV	=	Assumed Protection Value in dB
H	=	Attenuation for high frequencies
M	=	Attenuation for medium frequencies
L	=	Attenuation for lower frequencies
SNR	=	Single Number Rating

PZT GmbH • An der Junkerei 48 F • D-26389 Wilhelmshaven • EU-Identification Number 1974

Warning

Keep this product away from small children. This packaging has small parts that could be swallowed. When not in use, please store the earplugs in the supplied box. Do not insert any items in the filter, as they may damage the filter. If you are in a noisy environment, please use the NOIZEZZ hearing protection at all times. Failure to follow the procedures outlined in this manual may affect the life and the functioning of the earplugs. NOIZEZZ B.V. cannot be held liable for this.

Instructions: how to wear NOIZEZZ?

1. Take the XL earplugs between your thumb and index finger.

2. Adjust the earplug and determine when you have the right size, from large to small.

3. Place the filter in the earplug

4. Ready for use

Determine the right size earplug

The filters are in the XL size plugs. Take these plugs in size NOIZEZZ XL (largest size, the dimensions are on the plug) from the packaging. Take the handle of the earplug between your thumb and your index finger and ensure that the curved side is pointing up with the handle downwards.

Press the earplug with the side of the three ridges into the ear canal. At the same time feel if the earplug fits. If it doesn't fit, take a smaller size and repeat the procedure. Continue until the approach with the tab closes off the external ear canal.

CAUTION! Always start with size XL and each time take one size smaller until the right size has been found. Repeat this procedure for the other ear.

Removing NOIZEZZ

Remove the NOIZEZZ from the ear by gripping the handle with your thumb and index finger and pulling it from the ear.

Inserting and replacing of the noise filters

Press the filters in the earplug of the chosen size, from the side of the handle. Leave the head of the filter protrude from the earplug. Your NOIZEZZ are now ready for use. Squeeze the earplug to replace the filter.

Warranty

This product comes with a two-year warranty from the date of purchase. Any warranty claims must be made with the original receipt and at the NOIZEZZ dealer where the product was purchased. The warranty does not apply to damage caused by improper use or by normal wear. Will you be using the plugs every day? Please choose custom hearing protection.

Cleaning and maintenance

For an optimal functioning of your NOIZEZZ hearing protection we recommend to clean the earplugs regularly. Remove any ear wax with a soft cloth or tissue. Use lukewarm water and mild soap to clean the outside of your hearing protection.

Regularly check your hearing protection to ensure their optimal functioning. Their functioning might be affected if the earplugs are not placed properly in the ear, or if the filters are not placed properly in the earplug. The earplugs may become less effective due to improper maintenance. Several chemical substances may have a negative effect on the product. Additional information can be obtained from the manufacturer.

NOIZEZZ Premium hearing protection is reusable because it can be cleaned after use. To clean the earplugs you can use the NOIZEZZ Cleaning Spray, for sale at a NOIZEZZ point of sale or in the webshop.

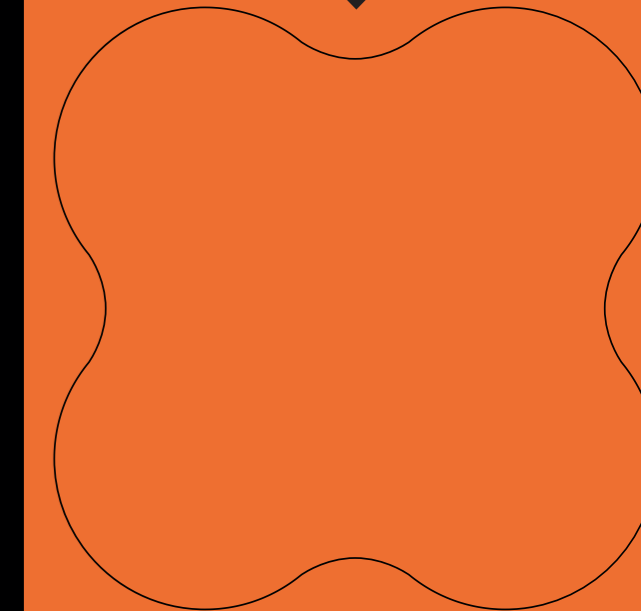
Please see www.noizezz.com for all points of sale.

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> Unique hearing protection equipment in 4 sizes for musicians and music listeners. Flat filter, which makes for an excellent music experience when rehearsing, performing or going to a gig. High wearing comfort thanks to its excellent ergonomic shape. Copper, brass and electric. Performing, clubs, festivals.

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noizezz

Read the instructions before use and retain for future reference.
For more information please contact:
NOIZEZZ B.V. E-mail: info@noizezz.com

 EN 352-2:2002
Manufactured by NOIZEZZ B.V.
Hoofddorp, Netherlands. WWW.NOIZEZZ.COM
Nominal diameter: S Ø 8 • M Ø 9 • L Ø 10 • XL Ø 11 mm

SNR 15 dB H 12 dB M 15 dB L 15 dB

EAN code:
8718247340378

70007