Nederlands

Demping

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APV	=	Assumed Protection Value in dB
H	=	Attenuation for high frequencies
M	=	Attenuation for medium frequencies
L	=	Attenuation for lower frequencies
SNR	=	Single Number Rating

PZT GmbH • An der Junkerei 48 F • D-26389 Wilhelmshaven • EU-Identification Number 1974

Warning

Keep this product away from small children. This packaging has small parts that could be swallowed. When not in use, please store the earplugs in the supplied box. If you are in a noisy environment, please use the NOIZEZZ earplugs for sleeping at all times. Failure to follow the procedures outlined in this manual may affect the life and the functioning of the earplugs. NOIZEZZ B.V. cannot be held liable for this.

Instructions: how to wear NOIZEZZ?

1. Take the XL earplugs between your thumb and index finger.

2. Adjust the earplug and determine when you have the right size, from large to small.

3. Ready for use

Determine the right size earplug

Take the plugs in size NOIZEZZ XL (largest size, the dimensions are on the plug) from the packaging. Take the handle of the earplug between your thumb and your index finger and ensure that the curved side is pointing up with the handle downwards.

Press the earplug with the side of the three ridges into the ear canal. At the same time feel if the earplug fits. If it doesn't fit, take a smaller size and repeat the procedure. Continue until the approach with the tab closes off the external ear canal.

CAUTION! Always start with size XL and each time take one size smaller until the right size has been found. Repeat this procedure for the other ear.

Removing NOIZEZZ

Remove the NOIZEZZ from the ear by gripping the handle with your thumb and index finger and pulling it from the ear.

Warranty

This product comes with a two-year warranty from the date of purchase. Any warranty claims must be made with the original receipt and at the NOIZEZZ dealer where the product was purchased. The warranty does not apply to damage caused by improper use or by normal wear. Will you be using the plugs every day? Please choose custom earplugs for sleeping.

Cleaning and maintenance

For an optimal functioning of your NOIZEZZ earplugs for sleeping we recommend to clean the earplugs regularly. Remove any ear wax with a soft cloth or tissue. Use lukewarm water and mild soap to clean the outside of your earplugs for sleeping.

Regularly check your earplugs to ensure their optimal functioning. Their functioning might be affected if the earplugs are not placed properly in the ear. The earplugs may also become less effective if the earplugs are not properly maintained. Several chemical substances may have a negative effect on the product. Additional information can be obtained from the manufacturer.

NOIZEZZ Premium earplugs for sleeping is reusable because it can be cleaned after use. To clean the earplugs you can use the NOIZEZZ Cleaning Spray, for sale at a NOIZEZZ point of sale or in the webshop.

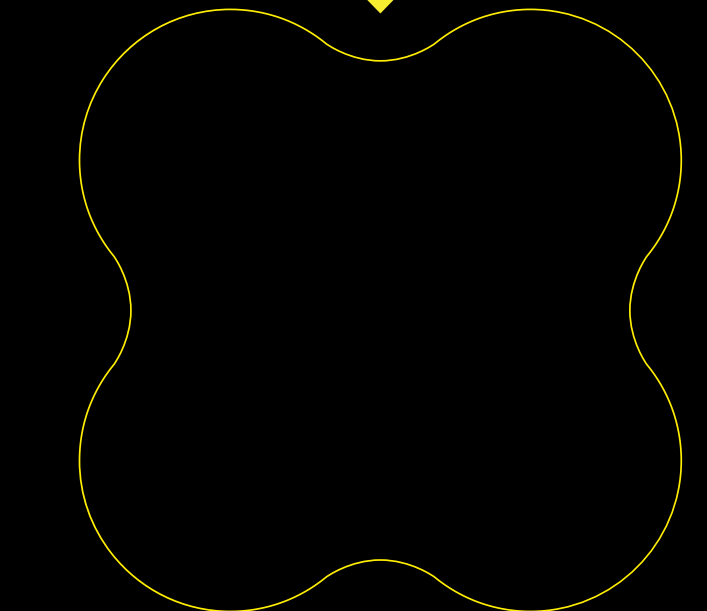
Please see www.noizezz.com for all points of sale.

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NOIZEZZ B.V. E-mail: info@noizezz.com
Manufactured by NOIZEZZ B.V.