NEDERLANDS

ENGLISH

## Demping

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Measured value, dB    H 19,5    M 17,3    L 13,7    SNR 19,1

APV = Assumed Protection Value in dB  
H = Attenuation for high frequencies  
M = Attenuation for medium frequencies  
L = Attenuation for lower frequencies  
SNR = Single Number Rating

## Warning

Keep out of reach of children. This package contains small parts that could pose a choking hazard if swallowed. If not used, we recommend that you keep the earplugs in the supplied can. Do not insert objects into the filter, as this may damage it. Use the NOIZEZZ earplugs at all times when in a noisy environment. Failure to follow the instructions provided in this manual may affect the lifespan and performance of the earplugs. NOIZEZZ B.V. cannot be held responsible for this.

## Instructions : How to place the ear plugs?

### Step 1: Determine the correct size

Take the NOIZEZZ XL (largest size; size is specified on the plug) out of the packaging. Grasp the tab between your thumb and forefinger and make sure that the curved side is facing up with the tab down (see Figure 1 for the left ear and Figure 2 for the right ear).

Insert the earplug until the eartip with the flanges fits comfortable in the ear canal (see Figure 3). Then test the fit, if too big, choose a smaller size and repeat the steps above until the eartip with the flanges is sealed in the ear canal entrance (Figure 4).

Note! Always start with the XL size and keep test fitting a smaller size until you have the right size. Repeat the steps for your other ear.

### Step 2: Removing the ear plugs

Remove the NOIZEZZ from your ear by holding the tab between your thumb and forefinger and pulling it out of your ear.

1. Left ear

2. Right ear

3. Insert ear plug

4. Ear plug in ear

5. Insert filter

6. Ready for use

### Step 3: Insert music filters

Press the filters from the tab side into the earplug of your size (see Figure 5). The head of the filter must remain visible. Your NOIZEZZ are now ready for use.

## Warranty

This product has two years of warranty from the date of purchase. The warranty claim is only valid with the original receipt and runs through the NOIZEZZ dealer. Warranty does not cover damage caused by improper or incorrect use, or normal wear. For daily use, we recommend customized hearing protection.

## Cleaning and maintenance

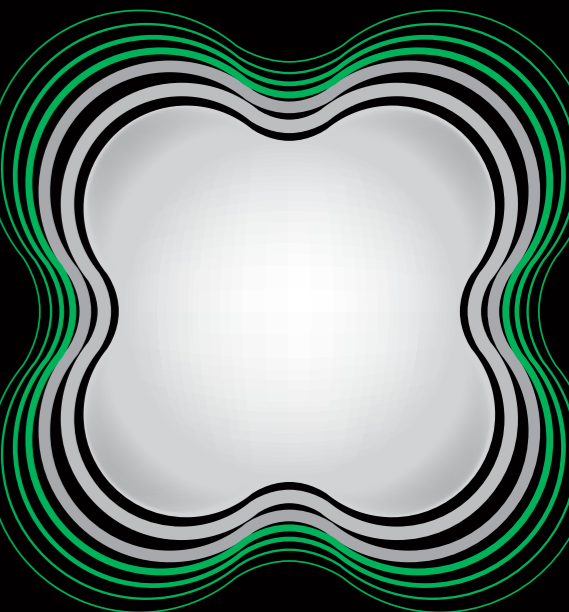
For peak performance, we recommend that you regularly clean your NOIZEZZ earplugs.

Use a soft cloth or tissue to remove earwax. Use lukewarm water and mild soap to clean the outside of your earplugs.

To maintain peak performance, regularly check your earplugs. If not worn properly or if the filters are not inserted correctly, the earplugs will not work effectively. Poor maintenance will also affect the earplugs' performance. Do not clean with harsh chemicals as they may have an adverse effect on the product. Additional information can be requested from the manufacturer.

NOIZEZZ universal protective earplugs are reusable because they can be cleaned after use.

For cleaning the earplugs, you can use NOIZEZZ Cleaning Spray, available at NOIZEZZ sales points or the webshop. Visit the website to see where our salespoints are located.

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Hearing protection with special filters for: entertainment (disco / dancing / concerts), catering staff, teachers in gymnasiums, swimming pools and children's creches. Unique filters preserve speech intelligibility. Very comfortable, thanks to the unique ergonomic shape.

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Read the instructions before use and retain for future reference.  
For more information please contact:

NOIZEZZ B.V.

Phone +31 (0)23 554 40 10

E-mail: [info@noizezz.com](mailto:info@noizezz.com)

Tested at CRITT France  
No:0501-123715905100106  
EN 352-2:2002

Developed and produced by  
NOIZEZZ B.V. Hoofddorp, Netherlands.  
[WWW.NOIZEZZ.COM](http://WWW.NOIZEZZ.COM)

70012